USER GUIDE

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Limbs & Things Ltd

of

Sussex Street, St. Philips, Bristol, BS2 0RA

Declare under our sole responsibility that the product described below:

Brand and Model Name	Limbs & Things Clean Bleed™ Mat, Part number: 70400
-----------------------------	--

Has been tested and is in conformity with the following standards:

<u>SUBJECT</u>	<u>STANDARD</u>
Electrical Safety	LVD Directive – 2014/35/EU EN 61010:2010 + A1:2019: Safety requirements for electrical equipment for measurement, control, and laboratory use. General requirements
Electromagnetic Compatibility	EMC Directive – 2014/30/EU EN61000-6-3:2007 + A1:2011 Electromagnetic compatibility (EMC). Generic standards. Emission standard for residential, commercial and light-industrial environments Conducted Emissions: EN61000-6-3:2007 + A1:2011 Radiated Emissions: EN61000-6-3:2007 + A1:2011 Voltage Fluctuations: EN61000-3-3:2013 Immunity: EN61000-6-1:2007: EN61000-4-4: 2012 EN61000-4-5: 2014
Radio performance	N/A
RoHS	RoHS - Directive 2011/65/EU, as amended (by Directive 2015/863)

I hereby declare that the equipment named above has been designed to comply with the relevant sections of the above referenced Standards.

Signed Nick Hull
Position Managing Director
Location Limbs & Things, Bristol, England

Signature 

Date 19 May 2021

DECLARATION OF CONFORMITY

In accordance with UK Government Guidance

We at

Limbs & Things Ltd

of

Sussex Street, St. Philips, Bristol, BS2 0RA

Declare under our sole responsibility that the product described below:

Brand and Model Name	Limbs & Things Clean Bleed™ Mat, Part number: 70400
----------------------	---

Has been tested and is in conformity with the following standards:

<u>SUBJECT</u>	<u>STANDARD</u>
Electrical Safety	LVD Directive – 2014/35/EU EN 61010:2010 + A1:2019: Safety requirements for electrical equipment for measurement, control, and laboratory use. General requirements
Electromagnetic Compatibility	EMC Directive – 2014/30/EU EN61000-6-3:2007 + A1:2011 Electromagnetic compatibility (EMC). Generic standards. Emission standard for residential, commercial, and light-industrial environments Conducted Emissions: EN61000-6-3:2007 + A1:2011 Radiated Emissions: EN61000-6-3:2007 + A1:2011 Voltage Fluctuations: EN61000-3-3:2013 Immunity: EN61000-6-1:2007: EN61000-4-4: 2012 EN61000-4-5: 2014
Radio performance	N/A
RoHS 3	RoHS - Directive 2011/65/EU, as amended (by Directive 2015/863)

I hereby declare that the equipment named above has been designed to comply with the relevant sections of the above referenced Standards.

Signed Nick Hull
Position Managing Director

Signature 
19 May 2021

We at
Limbs & Things Ltd
of

Sussex Street, St. Philips, Bristol, BS2 0RA

Declare under our sole responsibility that the product described below:

Brand and Model Name	Limbs & Things Clean Bleed™ Mat, Part number: 70400
-----------------------------	--

Has been tested and is in conformity with the following standards:

Technical regulations	FCC CFR 47 Part 15B
Test laboratory	Kiwi Electrical Compliance Unit 8 Woodfieldside Business Park Pontllanfraith Blackwood United Kingdom NP12 2DG
Test report number	R21-5433
Test report date	12 TH March 2021

Signed Nick Hull
Position Managing Director
Location Limbs & Things, Bristol, England

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©2021Limbsandthings
Nihon Light Service, Inc.