


Adam, Rouilly

ANATOMICAL MODELS, SIMULATORS AND CHARTS FOR CLINICAL SKILLS TRAINING

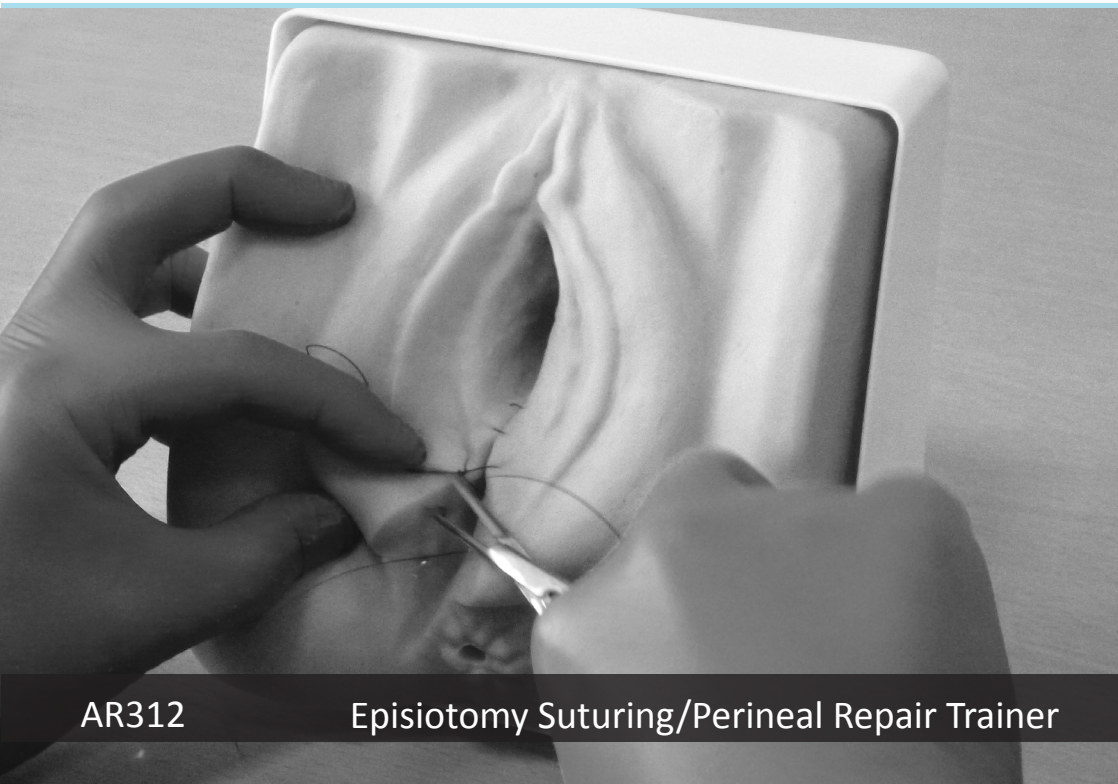

AR312

Episiotomy Suturing/Perineal Repair Trainer

Instruction Manual

Thank you for purchasing this AR312 Episiotomy Suturing/Perineal Repair Trainer.

Please read this instruction manual carefully and retain it for future reference.

Skills

- Repair of a right mediolateral incision
- Handling and knot tying

Features

- Accurate, life size anatomy with all relevant landmarks
- Improved material gives highly realistic feedback during suturing
- Open rectum for digital assessment and verification that the bowel has not been sutured
- Suited to deep muscle, subcutaneous and superficial suturing techniques
- Reinforced layer allows for repeated procedures to be performed
- Suction feet for easy, stable securement to work surface

Safety and Precautions



After each use, sutures must be removed from the model. Leaving the model sutured for prolonged periods may cause damage, decrease durability and invalidate your guarantee.



Remove sutures very carefully after each use. If instruments are used, be certain not to cut into or tear the skin of the model.




This simulator is a non-serviceable item and will need to be disposed of, in accordance with local policy, after a period of use.




Please treat the model with the same care you would a patient.

Parts and Configuration

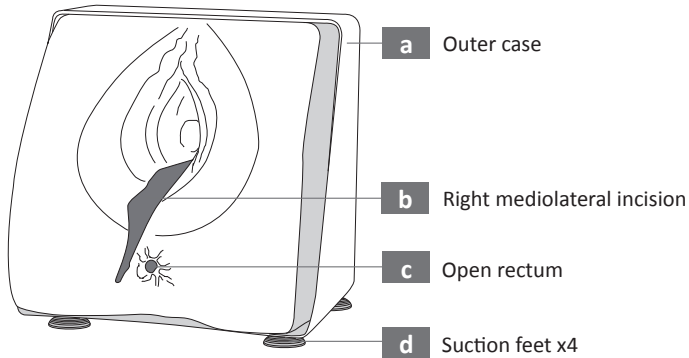


Before Use

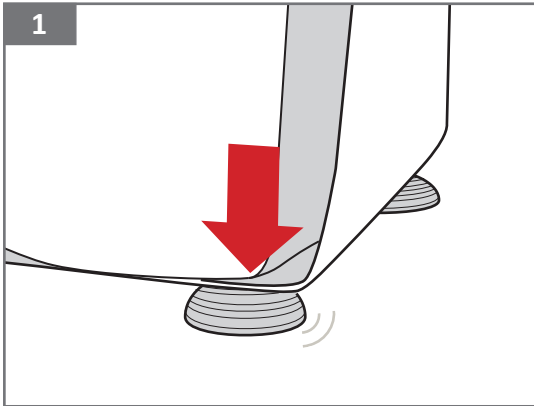

Place the model on a stable, flat surface.

Secure in place by using all 4 suction feet to fix to the surface.

After Use

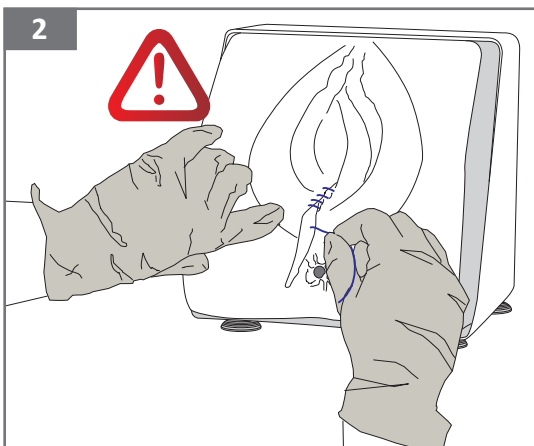

After training, sutures must be removed.

Remove sutures carefully, being certain not to rip or tear the skin of the model.



Leaving the model sutured for prolonged periods may cause damage, decrease durability and invalidate your guarantee.

Adam,Rouilly

Adam,Rouilly has over 95 year's experience in providing quality medical models and simulators.

Should you require any further information please contact our Sales Department who will be pleased to help with your enquiry.

Adam,Rouilly Limited
Castle Road
Eurolink Business Park
Sittingbourne
Kent
United Kingdom
ME10 3AG

T: +44(0)1795 471378
F: +44(0)1795 479787
E: sales@adam-rouilly.co.uk
www.adam-rouilly.co.uk